

# HOSPAL<sup>®</sup> - AERO

Aldehyde-free Aerosol Disinfectant for Fumigation

Bactericidal  
Fungicidal  
Virucidal

Aldehyde  
Free

One-shot /  
controlled  
release  
actuator



**PSK PHARMA PRIVATE LIMITED**


121/22 Mahajenahalli, Shimoga Road, Harihar - 577601, Karnataka, India  
Tel: +91 96060 69014 | e-mail: [weprotect@pskpharma.com](mailto:weprotect@pskpharma.com) | [www.pskpharma.com](http://www.pskpharma.com)



View Product Video



Download Test Reports

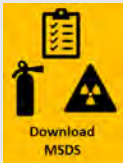

Download MSDS



ISO 13485 Medical Devices  
Certified QMS


PSK/MKT/MPL - Rev. 15 - dated 24.06.2022



WHO-GMP Certified

# HOSPAL<sup>®</sup> - AERO

## Aldehyde-free Aerosol Disinfectant for Fumigation

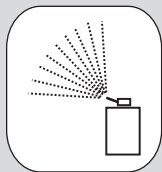

Bactericidal  
Fungicidal  
Virucidal

Aldehyde  
Free

One-shot /  
controlled  
release  
actuator

### APPLICATIONS:

For effective fumigation of areas including:

- Operation Theaters
- Wards
- Labs
- Cleanrooms
- Dental clinics
- Air conditioning ducts
- Ambulances


### COMPOSITION:

Each 10 mg contains:


Nimba (Azadirachtha Indica) 0.1 mg extracted in Ethyl Alcohol. Other ingredients: n-Butane, Isobutane, Propane, Didecyldimethyl Ammonium Chloride, Excipients q.s.

### DIRECTIONS FOR USE:

One 100 g can is recommended for fumigating 2,500 cubic feet of room volume

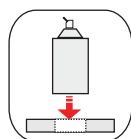


1. Shake the can well and push the bottom of the can firmly into the hole of the cardboard baseplate provided. Ensure that the bottom of the can is flush with the bottom surface of the baseplate.

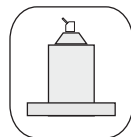

2. Place the can with baseplate on a flat surface in a corner of the room to be fumigated. Open the plastic cap of the aerosol can.




3. Firmly twist the actuator clockwise, keeping the nozzle pointed away from you. Ensure that spray is pointed towards the centre of the room. Exit the room while the device is in operation

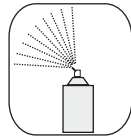

4. The can will complete discharging in about one minute. Do not ventilate the room; slow down evaporation by switching off fans and AC.



5. Keep the room closed and wait for at least one hour for fogging action to complete. Ventilate before entering the room.

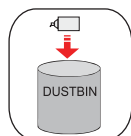

6. Discard the empty can, cap and baseplate appropriately

### SALIENT FEATURES:

- Aldehyde-free, non-toxic formulation
- Broad spectrum Bactericidal, Fungicidal, Virucidal activity
- Proven efficacy as per BS EN 1276: 2009 demonstrating > Log 5 efficacy in 2 minutes against *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Escherichia coli*, *Bacillus subtilis*, *Clostridium perfringens*, *Candida albicans*, MS2 Bacteriophage (Surrogate of Polio Virus)
- Contents can be discharged all at once for fumigation (one-shot) or in a controlled manner for surface disinfection



### PACKING:

100 g Aerosol Spray Can  
with Cardboard Base Plate