

# 2-Propanol & Benzalkonium Chloride Solution

# **FUMISPRAY®-RAPID**

# FOR DISINFECTION OF SURFACES & SENSITIVE **EQUIPMENTS**

# **ALDEHYDE FREE!** DOES NOT CONTAIN ALDEHYDES

### Each 100 gms contains:

2-Propanol IP 63 Gms Benzalkonium Chloride 0.025 Gms (As Benzalkonium Chloride Solution IP)

#### Salient Features:

- Contains Isopropyl Alcohol 63% by weight
- Aldehyde-free formulation
- Broad spectrum antimicrobial
- Dries without visible residue
- Ideal for disinfection of electronic equipment in OTs, AKD units, ICU, NICU, Recovery
- Room, Medical Laboratories, etc.

## Packaging:

• Provided with convenient @ å sprayer with 2 nozzle options - jetÁ •] ¦æ and mist spray



## Areas of Application: **Equipment Disinfection:**

- Electronic devices, imaging machines, respirators, ventilators, cardiac monitors, emergency trolleys, CT scanners, haemodialysis systems, operation tables, lights, etc.
- Emergency disinfection of operation theaters in between surgeries. ICU. NICU, ICCU, etc.

## **Directions for Use - Equipment Disinfection:**

- Set the spray nozzle at "Spray" position and keeping the nozzle around 25 cm away from the equipment surface, spray Fumispray®-Rapid liberally.
- Allow the liquid to stay on the surface for 30 seconds and wipe away with a clean, lint-free cloth.











#### **PSK PHARMA PRIVATE LIMITED**



