

# FUMIFLOOR® - ADVANCE

## Aldehyde-free, Sporicidal Surface Disinfectant

### Bactericidal, Fungicidal, Virucidal, Sporicidal



**ALDEHYDE FREE!**  
DOES NOT CONTAIN ALDEHYDES



#### **SALIENT FEATURES:**

- Aldehyde-free - excellent safety profile
- Eradicates spores in just 30 minutes
- Formulated based on Quaternary Ammonium Compound with synergistic compounds
- Excellent material compatibility - contains corrosion inhibitors
- Contains surfactant for cleaning action
- Effective even with hard water

#### **COMPOSITION:**

Each 100 g Contains:

Didcyldimethyl Ammonium Chloride 7.0 g

Fragrance, Excipients q.s.



#### **DILUTION:**

- Add 200 ml Fumifloor-Advance to 10 Litres of water to get 2% solution.
- Add 500 ml Fumifloor-Advance to 10 Litres of water to get 5% solution



#### **DIRECTIONS FOR USE:**

##### **FOR HIGH-LEVEL DISINFECTION:**

1. **Mopping:** Wet surfaces of tiled walls, trolleys, operation tables and floors evenly with 2% solution and allow surfaces to remain wet for at least 15 minutes.
2. **Fumigation:** Fumigate with 2% solution for at least 20 minutes, close room and allow to dry for 30 minutes.

##### **FOR SPORICIDAL ACTION:**

1. **Mopping:** Wet surfaces of tiled walls, trolleys, operation tables and floors evenly with 5% solution and allow surfaces to remain wet for at least 30 minutes.
2. **Fumigation:** Fumigate with 5% solution for at least 20 minutes, close room and allow to dry for 30 minutes.

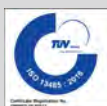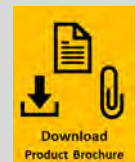

ISO 13485 Medical Devices  
Certified QMS

**PSK PHARMA PRIVATE LIMITED**

121/22 Mahajenahalli, Shimoga Road, Harihar - 577601, Karnataka, India

Tel: +91 96060 69014 | e-mail: [weprotect@pskpharma.com](mailto:weprotect@pskpharma.com) | [www.pskpharma.com](http://www.pskpharma.com)



WHO-GMP Certified

PSK/MKT/MPL - Rev. 19 - dated 19.01.2024