



HOSPAL[®] LINEN DISINFECTANT CONCENTRATE

Cleanses & Disinfects



ALDEHYDE FREE!
DOES NOT CONTAIN ALDEHYDES



Composition:

Each 100 g Contains:
Didecyldimethyl Ammonium Chloride 7.0 g
Fragrance, Excipients q.s.



Salient Features:

- Aldehyde-free - excellent safety profile
- Formulated based on Quaternary Ammonium Compound with synergistic compounds
- Excellent material compatibility - contains corrosion inhibitors
- Contains surfactant for cleaning action
- Effective even with hard water
- Helps reduce occurrence of bedsores when compared to traditional oxidizing agents

Hospital Linen Disinfectant concentrate is ideal for:

- Infected and contaminated linen
- Foul-smelling linen contaminated with urine and excreta
- Linen contaminated with blood, spit, sputum and mucous
- Clothing and linen used for patients with communicable disease
- Reusable surgical dressings
- Sheets, gowns, undergarments, floor mops, etc.

Directions for Use:

- **For Infected Linen (i.e., soiled with body fluids such as blood, sputum, mucous, urine, stools etc.):** Squeeze out the contaminants as much as possible, then pre-soak in 1% Hospel Linen Disinfectant solution for 30 minutes (i.e., 100 ml of Hospel Linen Disinfectant concentrate in 10 Litres of water)
- **For Non-infected Linen (sheets, gowns, blankets, covers, masks, etc.):** Pre-soak in 0.5% Hospel Linen Disinfectant solution for 30 minutes (i.e., 50 ml of Hospel Linen Disinfectant concentrate in 10 Litres of water)

Note:

1. Do not use with any other cleansing agents
2. Use in final rinse cycle of washing machine for best results



PSK PHARMA PRIVATE LIMITED

121/22 Mahajenahalli, Shimoga Road, Harihar - 577601, Karnataka, India

Tel: +91 96060 69014 | e-mail: weprotect@pskpharma.com | www.pskpharma.com



ISO 13485 Medical Devices
Certified QMS



WHO-GMP Certified